


Azure™ Ti-Base-C

Multi-Platform Prosthetic Solutions

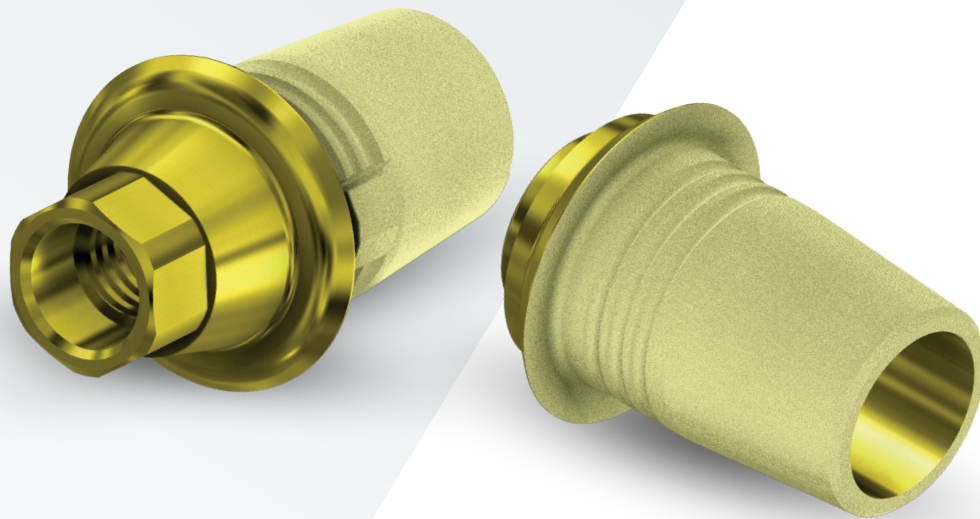


*Your Connection
for Every Connection*

azure™


Ti-Base-C

Re-engineered to Perfection

Enhanced surface properties and features for an improved workflow experience.

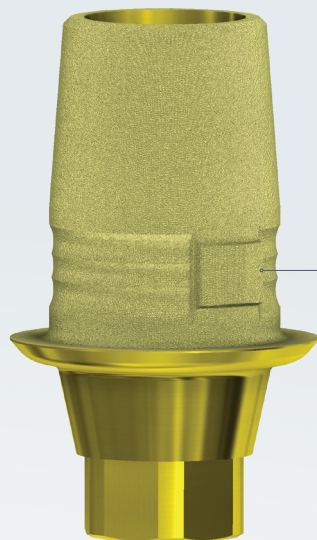

ENGAGING OR
NON-ENGAGING
OPTIONS

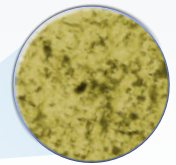

*Microscopic View


Patented Surface Technology

The Ti-Base post features a patented surface technology for optimal retention of the prosthesis.



Azure Ti-Base-C includes final screw

Identical to the implant brand screw.

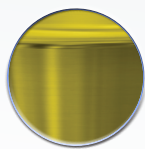

Gold-anodized Surface

Gold-anodized surface for improved aesthetics as compared to a traditional silver colored abutment or Ti-Base.



Multiple Gingival Heights

Available in various collar heights for restorative flexibility.

Digital Workflow

CEREC®/inLab Systems

- 1 Scan the Azure Ti-Base-C intra-orally, or on the working model.

Use either the relevant Sirona Scanpost or the S/L Scan Bodies on the Azure Ti-Base-C.



Order from a third party distributor using Sirona Scan Body manufacturer codes:

Bluecam S (6431295), **Bluecam L** (6431303), **Omnicaam S** (6431311), **Omnicaam L** (6431329), **InCoris ZI meso F0.5 S** (6231802), **InCoris ZI meso F0.5 L** (6231810), **InCoris ZI meso F2 S** (6231828), **InCoris ZI meso F2 L** (6231836)

- 2 Select the CEREC/inLab software reference.

Match the implant connection and platform compatibility.




- 3 Follow the Sirona® CAD software instructions.
Design the definitive restoration for your patient.

- 4 Use the milling zirconia blocks to mill your restoration.



- 5 Cement the zirconia restoration to the Ti-Base-C.

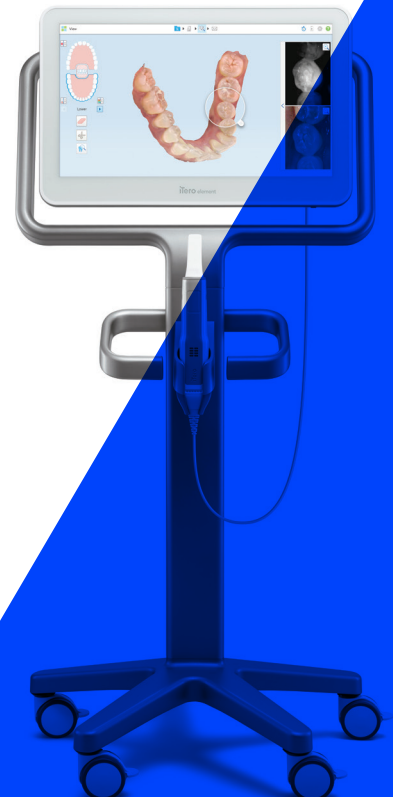

AZURE TI-BASE-C QUICK REFERENCE GUIDE

Azure System	Sirona Scan Body Size¹	Sirona Scanpost Reference	Implant Platform				
				GH	Non-engaging²	Engaging	
DAEV-Conical (Astra Tech Implant System™ EV)	S	AT EV 3.6 GH1 S	3.6 mm (P)	1.0 mm	ARDAEV-TBCNH361	ARDAEV-TBCH361	
	L	AT EV 4.2 GH1 L	4.2 mm (P)	1.0 mm	ARDAEV-TBCNH421	ARDAEV-TBCH421	
		AT EV 4.8 GH1 L	4.8 mm (P)	1.0 mm	ARDAEV-TBCNH481	ARDAEV-TBCH481	
DATX-Internal (Astra Tech Osseospeed™)	L	AT OS 3.5/4.0 L	3.5/4.0 mm (P)	0.8 mm	ARDATX-TBCNH3508	ARZTSV-TBCH4508	
		AT OS 4.5/5.0 L	4.5/5.0 mm (P)	0.9 mm	ARDATX-TBCNH4509	ARDATX-TBCH4509	
BHI-Internal (BioHorizons® Internal)	L	BH 3.5 L	3.5 mm (P)	0.5 mm	ARBHI-TBCNH3505	ARBHI-TBCH3505	
		BH 4.5 L	4.5 mm (P)	0.5 mm	ARBHI-TBCNH4505	ARBHI-TBCH4505	
		BH 5.7 L	5.7 mm (P)	0.5 mm	ARBHI-TBCNH5705	ARBHI-TBCH5705	
CCO-Conical (Conelog®)	S	CONELOG 3.3/x	3.3 mm (P)	1.0 mm	ARCCO-TBCNH331	ARCCO-TBCH331	
		CONELOG 3.8/x	3.8/4.3 mm (P)	1.0 mm	ARCCO-TBCNH381	ARCCO-TBCH381	
	L	CONELOG 5.0/x	5.0 mm (P)	1.0 mm	ARCCO-TBCNH51	ARCCO-TBCH51	
DA-Internal (Dentspy Ankylos® C/X)	S	ANKN C/ GH1 S	C/X	1.4 mm	ARDA-TBCNH3514	ARDA-TBCH3514	
		ANKN C/ GH2 S	C/X	2.5 mm	ARDA-TBCNH3525	ARDA-TBCH3525	
NAC-Conical (NobelActive® and NobelReplace®)	L	NB A 4.5 L	3.5 mm (P)	0.8 mm	ARNAC-TBCNH3508	ARNAC-TBCH3508	
		NB A 5.0 L	4.3/5.0 mm (P)	0.8 mm	ARNAC-TBCNH4308	ARNAC-TBCH4308	
NB-External (Nobel Brånemark®)	L	NB B 3.4 L	3.4 mm (P)	0.6 mm	-	ARNB-TBCH3506	
		NB B 4.1 L	4.1 mm (P)	0.6 mm	-	ARNB-TBCH4106	
NRT-Trilobe (Nobel Replace Select™)	L	NB RS 3.5 L	3.5 mm (P)	0.4 mm	ARNRT-TBCNH3504	ARNRT-TBCH3504	
		NB RS 4.3 L	4.3 mm (P)	0.4 mm	ARNRT-TBCNH4305	ARNRT-TBCH4305	
		NB RS 5.0 L	5.0 mm (P)	0.4 mm	ARNRT-TBCNH504	ARNRT-TBCH504	
		NB RS 6.0 L	6.0 mm (P)	0.4 mm	ARNRT-TBCNH604	ARNRT-TBCH604	
OH-Conical (Osstem® TS/Hoissen ET)	L	O TS 3.5 L	Mini	1.0 mm	AROH-TBCNH31	AROH-TBCH31	
		O TS 4.0 L	Regular	1.0 mm	AROH-TBCNH41	AROH-TBCH41	
SBL-Conical (Straumann® Bone Level and Azure Tapered Bone Level)	L	S BL 3.3 L	3.3 mm (P)	0.7 mm	ARSBL-TBCNH3307	ARSBL-TBCH3307	
		S BL 4.1 L	4.1/4.8 mm (P)	0.7 mm	ARSBL-TBCNH4107	ARSBL-TBCH4107	
STL-Tissue (Straumann® Soft Tissue Level)	L	S SO 4.8 L	4.1/4.8 mm (P)	1.0 mm	ARSTL-TBCNH411	ARSTL-TBCH411	
		S SO 6.5 L	4.8/6.5 mm (P)	1.2 mm	ARSTL-TBCNH4812	ARSTL-TBCH4812	
ZC-Certain (ZimVie Certain®)³	S	B C 3.4 S	3.4 mm (P)	0.4 mm	ARZC-TBCNH3404	ARZC-TBCH3404	
	L	B C 4.1 L	4.1 mm (P)	0.4 mm	ARZC-TBCNH4104	ARZC-TBCH4104	
		B C 5.0 L	5.0 mm (P)	0.4 mm	ARZC-TBCNH505	ARZC-TBCH505	
ZEX-External (ZimVie External Hex)³	L	B 0 3.4 L	3.4 mm (P)	-	-	Coming soon	
		B 0 4.1 L	4.1 mm (P)	0.6 mm	-	ARZEX-TBCH4106	
		B 0 5.0 L	5.0 mm (P)	0.6 mm	-	ARZEX-TBCH506	
ZTSV-Internal (ZimVie TSV®)³	L	Z TSV 3.5 L	3.5 mm (P)	0.5 mm	ARZTSV-TBCNH3505	ARZTSV-TBCH3505	
		Z TSV 4.5 L	4.5 mm (P)	0.5 mm	ARZTSV-TBCNH4505	ARZTSV-TBCH4505	
		Z TSV 5.7 L	5.7 mm (P)	0.5 mm	ARZTSV-TBCNH5705	ARZTSV-TBCH5705	

¹To scan, use Dentsply Sirona® Scan Bodies: Bluecam S (6431295); Bluecam L (6431303); Omnicam S (6431311); Omnicam L (6431329); InCoris ZI meso F0.5 S (6231802); InCoris ZI meso F0.5 L (6231810); InCoris ZI meso F2 S (6231828); InCoris ZI meso F2 L (6231836)

²References can only be used with CEREC inLab software.

³Not available in all markets, please check with your local ZimVie representative.

Azure Product Codes								
	GH	Non-engaging²	Engaging	GH	Non-engaging²	Engaging	Azure Replacement Screw	Torque
	3.0 mm	ARDAEV-TBCNH363	ARDAEV-TBCH363	-	-	-	ARDAEV-SHTI12736	25 Ncm
	3.0 mm	ARDAEV-TBCNH423	ARDAEV-TBCH423	-	-	-	ARDAEV-SHXTI2742	
	3.0 mm	ARDAEV-TBCNH483	ARDAEV-TBCH483	-	-	-	ARDAEV-SHTI12748	
	3.0 mm	ARDATX-TBCNH353	ARDATX-TBCH353	-	-	-	ARDATX-SHTI1274	20 Ncm
	3.0 mm	ARDATX-TBCNH453	ARDATX-TBCH453	-	-	-	ARDATX-SHTI12745	25 Ncm
	3.0 mm	ARBHI-TBCNH353	ARBHI-TBCH353	-	-	-	ARBHI-SHXTI127U	30 Ncm
	3.0 mm	ARBHI-TBCNH453	ARBHI-TBCH453	-	-	-		
	3.0 mm	ARBHI-TBCNH573	ARBHI-TBCH573	-	-	-		
	2.1 mm	ARCCO-TBCNH3321	ARCCO-TBCH3321	-	-	-	Coming soon	20 Ncm
	2.1 mm	ARCCO-TBCNH4821	ARCOO-TBCH3821	-	-	-	Coming soon	
	2.1 mm	ARCCO-TBCNH21	ARCOO-TBCH521	-	-	-	Coming soon	
	-	-	-	-	-	-	ARDA-SHTITC134	15 Ncm
	-	-	-	-	-	-		
	3.0 mm	ARNAC-TBTCNH353	ARNAC-TBCH353	-	-	-	ARNAC-SUGTI35	35 Ncm
	3.0 mm	ARNAC-TBCNH433	ARNAC-TBTCH433	-	-	-	ARNAC-SUGTI43	
	1.5 mm	ARNB-TBCNH3515	ARNB-TBCH3515	3.0 mm	ARNB-TBCNH353	ARNB-TBCH353	ARNAC-SUGTI35	35 Ncm
	1.5 mm	ARNB-TBCNH4115	ARNB-TBCH4115	3.0 mm	ARNB-TBCNH413	ARNB-TBCH413	ARNB-SUGTI41	
	1.5 mm	ARNRT-TBCNH3515	ARNRT-TBCH3515	3.0 mm	ARNRT-TBCNH353	ARNB-TBCH413	ARNRT-SUGTI35	35 Ncm
	1.5 mm	ARNRT-TBCNH4315	ARNRT-TBCH4315	3.0 mm	ARNRT-TBCNH433	ARNRT-TBCH353	ARNRT-SUGTI45	
	1.5 mm	ARNRT-TBCNH515	ARNRT-TBCH515	3.0 mm	ARNRT-TBCNH53	ARNRT-TBCH433		
	-	-	-	-	-	ARNRT-TBCH53		
	3.0 mm	AROH-TBCNH33	AROH-TBCH33	-	-	-	AROH-SHTI12233	20 Ncm
	3.0 mm	AROH-TBCNH43	AROH-TBCH43	-	-	-	AROH-SHXTI12240	30 Ncm
	3.0 mm	ARSBL-TBCNH333	ARSBL-TBCH333	-	-	-	ARSBL-STRXTI33N	35 Ncm
	3.0 mm	ARSBL-TBCNH413	ARSBL-TBCH413	-	-	-	ARSBL-STRXTI44	
	-	-	-	-	-	-	ARSTL-STRXTI44	35 Ncm
	-	-	-	-	-	-		
	1.5 mm	ARZC-TBCNH3415	ARZC-TBCH3415	3.0 mm	ARZC-TBCNH343	ARZC-TBCH343	ARZC-SHTI1203	20 Ncm
	1.5 mm	ARZC-TBCNH4115	ARZC-TBCH4115	3.0 mm	ARZC-TBCNH413	ARZC-TBCH413		
	1.5 mm	ARZC-TBCNH515	ARZC-TBCH515	3.0 mm	ARZC-TBCNH53	ARZC-TBCH53		
		Coming soon	Coming soon		Coming soon	Coming soon	ARZEX-SHXTI1203	20 Ncm
	1.5 mm	ARZEX-TBCNH4115	ARZEX-TBCH4115	3.0 mm	ARZEX-TBCNH413	ARZEX-TBCH413		
	1.5 mm	ARZEX-TBCNH515	ARZEX-TBCH515	3.0 mm	ARZEX-TBCNH53	ARZEX-TBCH453		
	1.5 mm	ARZTSV-TBCNH3515	ARZTSV-TBCH3515	3.0 mm	ARZTSV-TBCNH353	ARZTSV-TBCH353	ARZTSV-SHXTI1274	30 Ncm
	1.5 mm	ARZTSV-TBCNH4515	ARZTSV-TBCH4515	3.0 mm	ARZTSV-TBCNH453	ARZTSV-TBCH453		
	1.5 mm	ARZTSV-TBCNH5715	ARZTSV-TBCH5715	3.0 mm	ARZTSV-TBCNH573	ARZTSV-TBCH573		

azure[™] | A ZimVie Brand

For more information, visit azuredental.com

Unless otherwise indicated, as referenced herein, all trademarks and intellectual property rights are the property of ZimVie Inc. or an affiliate; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. CEREC is a trademark and/or registered trademark of DENTSPLY SIRONA Implants. Azure MPS abutments, components, and tools are manufactured by Terrats Medical S.L. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie. ZV2370 REV B 01/25 ©2025 ZimVie Inc. All rights reserved.

